

## PROCESSING PRECI-BUT

1. Determine the path of insertion of the prosthesis.
2. Seat the **1902** paralleling mandrel on the **1904** male and place it into the surveyor.
3. Attach the male with sticky wax to the post-coping or to the bar.
4. Invest, burnout, and cast.
5. Cement the post-copings in the mouth and take impression.
6. Place the **1906** model analogues in the impression and pour a model in stone.
7. Press the **1907** processing female over the model analogue and make a refractory model.
8. Place the **1908** retention basket pattern over the replica of the processing female and incorporate it into the wax pattern of the cast metal frame.
9. Invest, burnout, and cast.
10. The retention basket is now cast in metal and will considerably reinforce the acrylic resin covering the attachment.
11. Finish the frame.
12. Seat the original **1903** female with space maintainer onto the model analogue and finish the prosthesis.