

## PROCESSING PRECI-POST

1. Prepare the root canal with the standardized **2004G/2004B** yellow or blue root canal reamer and place the corresponding **2002G/2002B** yellow or blue opaque impression post in the canal.
2. Take impression. The posts are provided with retentions for a proper fit in the impression material.
3. Pour a working model.
4. Whether the impression is plated or an acrylic resin model is made, the opaque impression post does not bond with any of these materials.
5. Replace in the laboratory the impression post with the transparent **2003G/2003B** yellow or blue laboratory post. Reduce to its correct length and make sure that the top offers a little retention for wax.
6. Make the wax pattern, invest, burnout, and cast. The result is always an exactly fitting post-coping without soldering joint.