

PROCESSING PRECI-TUBIX - PRECI-MORTIX

1. Wax up the crowns or the post-copings.
2. Assemble the attachment and incorporate the paralleling mandrel into the surveyor.
3. Remove as much wax as necessary to incorporate the female into the abutment crown or the post-coping.
4. Position the attachment parallel to the path of insertion of the secondary part.
5. Secure the female with wax and carefully remove the male. Reduce the attachment after casting.
6. Carefully invest and cast.
7. Cut off the paralleling mandrel with a hot instrument. Press the male into the female and provide the "wing" of the male with some retention for wax.
8. Wax up the other crowns, post-copings, or pontics.
9. Remove the wax pattern with the male from the model. Invest, burnout, and cast.
10. Slightly polish the male in case of excessive friction.