

**REBASING/RELINING A PROSTHESIS WITH PRECI-BAR
(1000/1100 SERIES)**

Article numbers: **1100/MR/OR - 1100/MR/IN - 1001/MR/IN**
1103/MR/OR - 1103 MR/IN - 1002/MR/IN

1. Take a reline impression with the rider in the prosthesis.
2. Fill up the rider and the bar area with silicone.
3. Pour a stone model. Always use a reline flask or jig.
4. Reline as usual.
5. Finish and polish.
6. Activate the rider if necessary (see INFO 108).