

## PROCESSING THE PRECI-CLIP PLASTIC BAR PATTERN

1. Take a full arch impression.
2. Wax up and cast the crowns or the post-copings. With implant constructions, provide the implant analogues with cast-to cylinders.
3. Adjust the **1014/B** plastic bar pattern to the ridge anatomy. The bar pattern may be deformed at 40 °C.
4. Connect the bar pattern to the abutments with sticky wax or self-curing resin.
5. Invest and cast as usual. Use a hard alloy.
6. Finish the abutment retainers and the bar pattern, and reposition them on the model. Place the **1105/1106/1125/1126** rider with the space maintainer on the bar. Do not remove too much material from the bar during finishing and polishing.
7. Block out the entire bar area, leaving the riders for 2/3 free. Also cover the top of the bar to create resilience.
8. Also block out the lower part of the riders to allow for future activation.
9. Finish the prosthesis as usual.
10. After polymerization, remove the space maintainer and activate the riders if necessary (see INFO 107).