

WORKING DUMMIES/ANALOGUES

To prevent spring pins from being damaged during processing, working dummies with reduced friction or retention levels are available in the CEKA Attachment System. These dummies replace the original spring pin during fabrication of the removable partial or full denture.

Two different working dummies are available:

- **RE H 1:** for **REVAX M 2** spring pins **RE 0031**
Use the **RE H 5** laboratory key.
- **H 1:** for **REVAX M 3** spring pins **694 C**
Use the **RE H 5** laboratory key.

The spring pins are always supplied with a space maintainer.

Always replace the working dummy at the end of the working procedure with the original spring pin (see INFO 063). Do not forget to secure the spring pin with **CEKA BOND** (see INFO 069).

Attention: the working dummies are made of non-precious alloys.
They must **never** be used in the **patient's mouth**.